

Photography Tips TeleDermatology

Four pictures can be added to the TeleDermatology Consult:

1 overview picture and **3 close-ups**.

N.B.: make sure you take several pictures in order to select the best four on your pc.

Preparation

- **Lighting:** lots of dispersed lighting: daylight or TL-light
- **Background:** pale, non-translucent, contrasting the skin (e.g. blue, green, or white)
- **Positioning:** the object of photography needs to be flat against the background, try and fixate patient and camera.

The camera



1. Power button
2. Flash button
 - **Off** (overview picture)
 - **Automatic** (close-up)
3. Shutter button



Taking pictures

Place a measuring tape directly adjacent to the skin disorder to help the camera focus more easily and get sharper pictures.

Focus

Press the Shutter button halfway to focus and set exposure.

When the framing marks turn **[green]**, press the Shutter button completely down.

Note: when the green brackets appear do not move the camera. If you need to reposition the camera then let go of the shutter button.



When the notification **AF** is visible on the LCD-screen, the distance to the object is too short. Please take a bit more distance (at an interval of 5 cm) and try again.




Overview picture (1x)

- Flash off
- Distance +/- 50 cm

Close-up (3x)

- Flash off
- Distance > 10 cm (close as possible)

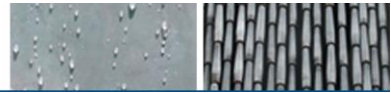

<p>Overview</p>		<p>Moved</p>		<p>Underexposed</p>		<p>Overexposed</p>	
<p>Close-up</p>		<p>Shiny background</p>		<p>Wrong background colour</p>		<p>Shadow</p>	
<p>Focused on background</p>							